



SUPERB D-RING

AN INTELLIGENT SOLUTION

Quality and functionality - here at ERKA. we constantly strive for perfection and apply this high standard to all of our products. To that end, our quality control is rigorous: each and every ERKA. GREEN CUFF is tested for proper operation - this is our ERKA. 100% Quality Check. In addition, all of our cuffs meet the requirements of the international AAMI and AHA guidelines. Biocompatible, water-resistant and entirely dermatologically safe: our cuff covers - made of a refined cotton blend fabric lined with polyurethane - can be perfectly cleaned and easily disinfected. This means maximum hygiene for patient and staff.

HOW CAN THE RISK OF INFECTION BE MINIMIZED FOR BOTH - THE PATIENT AND MEDICAL PROFESSIONALS?

The surest protection is the use of recyclable disposable cuffs. We offer a comprehensive range here. The intentional omission of PVC and DEHP in our products and optimal supply chain combine ecology and efficiency in the most impactful way to act responsibly.

WHICH ALTERNATIVE PRODUCTS CAN I USE INSTEAD OF THE RECYCLABLE DISPOSABLE CUFF?

All reusable ERKA GREEN CUFF blood pressure cuffs can be disinfected thoroughly and quickly because of their unique surface and materiality.

IS THERE EVEN MORE PROTECTION PROVIDED BY ERKA?

It is recommendable to use ERKA reusable cuffs with the matching ERKA hygiene protection stripes (Ref. 000.04100). This prevents a direct skin contact and minimizes therefore potential risks.

WHICH ERKA BLOOD PRESSURE MEASURING INSTRUMENTS ARE SUITED FOR SINGLE TUBE CUFFS?

Kobold Switch 2.0

DUE TO THE UNIQUE MEMBRANE-PRESERVING TWIN TUBE TECHNOLOGY THE FOLLOWING DEVICES ARE EQUIPPED WITH TWIN TUBE CUFFS:

Perfect Aneroid 48 Perfect Aneroid 56 Perfect Aneroid Klinik 48 Perfect Aneroid Klinik 56 Profi 48 Profi 56 Profi Klinik 48

FOR A MAXIMUM OF OPERATING CONVENIENCE ALL FIXED DEVICES ARE EQUIPPED WITH DOUBLE-TUBE CUFFS:

Erkameter 3000 Erkameter 3000 eco Vario

SUPERB

Superb stands for the uncompromising design free of PVC, DEHP and latex. Both - in the choice of materials and in the manufacturing process.

D-RING FASTENER

Convenient: when applied at the upper arm, the D-Ring forms a conical basic shape which optimizes the fit of the cuff during the measurement.

HOW CAN THE REUSABLE ERKA CUFF BE CLEANED?

The following ERKA GREEN CUFFS are equipped with a water resistant, easy to clean surface and therefore can be disinfected easily and thoroughly: Superb Rapid Superb rapid one piece Superb Hook Superb D-Ring Superb Home Smart rapid Smart rapid one piece

HOW CAN THE REUSABLE ERKA CUFF BE DISINFECTED?

All ERKA reusable cuffs may be disinfected thoroughly by spraying or wiping with tested approved disinfectants. Afterwards the surface is wiped with light pressure.

WHICH DISINFECTANTS ARE RECOMMENDED BY ERKA?

Fugaten® Spray Esemfix® from Schülke Promanum® pur from B.Braun Meliseptol® Foam pur from B.Braun

WHY ALL ERKA GREEN BLOOD PRESSURE CUFFS ARE SO PLEASANT TO USE?

The optimal comfort is determined by the combination of materials used and the complete absence of PVC , DEHP and similar substances. Furthermore, our materials are biocompatible and satisfy the requirements specified in DIN ISO 10993.

HOW TO PLACE THE ERKA CUFF CORRECTLY?

To avoid any error the cuff must be placed correctly. The brachial artery is clearly marked on all ERKA cuffs: it is a reliable indicator for positioning the cuff.

HOW TO CHOOSE THE CORRECT SIZE OF THE CUFF?

All ERKA GREEN CUFFs are clearly marked with a size range. One must choose a different cuff size if the arm circumference does not fit into one of the ranges.

Superb D-Ring	Size	Reference
Size 4	025.40881.	
Size 4	025.40882.	
Size 4	025.40891.	
Size 4	025.40892.	
Size 4	025.40893.	
Size 4	025.40894.	
Size 5	025.44881.	
Size 5	025.44882.	
Size 5	025.44891.	
Size 5	025.44892.	
Size 5	025.44893.	
Size 5	025.44894.	
Size 3	025.46881.	
Size 3	025.46882.	
Size 3	025.46891.	
Size 3	025.46892.	
Size 3	025.46893.	
Size 3	025.46894.	

Accessories

Size

Reference

Spare Parts

Legal Notes and Data Protection

ERKA.
Kallmeyer Medizintechnik GmbH & Co. KG
Im Farchet 15
83646 Bad Tölz

Geschäftsführer: Björn Kallmeyer

HRA 75980, Amtsgericht München
Umsatzsteuer-ID: DE 812 833 893

Kontakt:
Telefon: 0 80 41 / 80 09-0
Telefax: 0 80 41 / 80 09-39
eMail: made@erka.org
<http://www.erka.org>

Verantwortlich:
Björn Kallmeyer

Signatur / Stempel

HOW TO CHOOSE THE CORRECT SIZE OF THE CUFF?

