



ERKA. SWITCH 2.0 HOMECARE

EFFICIENCY AND PRECISION

Designed in close cooperation with medical professionals, blood pressure monitoring at a professional level is now possible at home. As a pioneer in blood pressure monitoring, we have developed the ERKA. Switch 2.0 Homecare with the expertise of 125 years. Now you can depend on this expertise at home too. Comfortable to handle and functional in its application: the ERKA. Switch 2.0 Homecare combines cost-efficient material selection with superb accuracy and thereby exceeds all the requirements of a modern blood pressure measuring device.

ERKA MANUAL SPHYGMOMANOMETERS ARE THE REFERENCE IN DURABILITY AND PRECISION FOR DECADES - WHY?

As early as the production phase, every single ERKA device undergoes no less than 49 quality and functional checks - including a final computer-controlled 100% final inspection. This was we guarantee that the user has the best blood pressure monitor at their fingertips for professional duties.

WHY ERKA MANUAL SPHYGMOMANOMETERS ARE SO PRECISE?

The use of a special tempered copper-beryllium membrane in combination with the filigree measuring mechanism ensures durability along with the consistently high precision.

HOW LONG IS THE APPROXIMATELY SERVICE LIFE OF AN ERKA MANUAL SPHYGMOMANOMETER?

The unique membrane-preserving twin tube technology - developed by ERKA - guarantees maximum durability even with daily use.

CAN THE ERKA TEST BE USED OPTIMALLY BY BOTH RIGHT- AND LEFTHANDED OPERATORS?

Yes, especially for lefthanded operators we have a left-hander device design.

DOES DELIVERY INCLUDE A STETHOSCOPE?

Every ERKA test blood pressure device is equipped with a high quality stethoscope from the Erkaphon family provides the best acoustics and accurate readings.

HOW IMPACT-SENSITIVE IS THE ERKA TEST?

The use of high quality materials, the modern double tube technology and the shock-resistant construction suitable for everyday use.

MEMBRANE

The use of a special tempered copper-beryllium membrane ensures durability along with the consistently high precision of the measuring device.

SHOCK-RESTISTANT

The device is shock-proof ? this feature is essential since normal measuring system errors indicated by impact are usually not obvious.

SCREW VALVE

The sensitive screw valve enables the user to take perfect measurements.

MODIFICATION

The device is available in both right-and left-handed models.

STETHOSCOPE

The high quality stethoscope from the Erkaphon family provides the best acoustics and accurate readings.

HOW CAN THE REUSABLE ERKA CUFF BE CLEANED?

All ERKA GREEN CUFFS are equipped with a water resistant, easy to clean surface and therefore can be disinfected easily and thoroughly.

HOW CAN THE REUSABLE ERKA CUFF BE DISINFECTED?

All ERKA reusable cuffs may be disinfected thoroughly by spraying or wiping with tested approved disinfectants. Afterwards the surface is wiped with light pressure.

WHICH DISINFECTANTS ARE RECOMMENDED BY ERKA?

Fugaten® Spray Esemfix® from Schülke Promanum® pur from B.Braun Meliseptol®
Foam pur from B.Braun

WHY ALL ERKA GREEN CUFF BLOOD PRESSURE CUFFS ARE SO PLEASANTLY TO WEAR?

All ERKA GREEN CUFFS are equipped with a water resistant, easy to clean surface and therefore can be disinfected easily and thoroughly.

HOW THE ERKA CUFF IS APPLIED PROPERLY?

To achieve optimum measuring results the cuff must be applied properly. So every single ERKA cuff is labeled with clearly indicated size and positioning instructions to ensure a proper selection and flawless application of the cuff.

HOW CAN I FIGURE OUT THE RIGHT CUFF SIZE?

Every ERKA GREEN CUFF blood pressure cuff is labeled with a clearly visible size range; please choose another cuff size if your arm circumference does not fit into this range.

Erka. Switch 2.0 homecare	Size	Reference
ERKA GreenCuff Superb D-Ring Cuff, grey	Size 4	292.40893.
ERKA GreenCuff Superb D-Ring Cuff, grey	Size 5	292.44893.
ERKA GreenCuff Superb D-Ring Cuff, grey	Size 3	292.46893.

Accessories

Size

Reference

Spare Parts

Pump Ball grey	000.20900
Measuring Vessel Switch 2.0 Comfort	000.30200

Legal Notes and Data Protection

ERKA.
Kallmeyer Medizintechnik GmbH & Co. KG
Im Farchet 15
83646 Bad Tölz

Geschäftsführer: Björn Kallmeyer

HRA 75980, Amtsgericht München
Umsatzsteuer-ID: DE 812 833 893

Kontakt:
Telefon: 0 80 41 / 80 09-0
Telefax: 0 80 41 / 80 09-39
eMail: made@erka.org
<http://www.erka.org>

Verantwortlich:
Björn Kallmeyer

Signatur / Stempel

HOW CAN I FIGURE OUT THE RIGHT CUFF SIZE?

